

# DreamWear **patient fitting guide**

Follow the instructions to get started fitting your full face, nasal or silicone pillows mask

## Full face

Using the full face sizing gauge, hold horizontally under the nose and select the cushion size based on where the outer edge of your nostrils and tip of your nose contact the gauge.

Note: Four cushion sizes (small, medium, large, medium-wide) are available. Please refer to your instructions for use for further fitting information.



1

The cushion opening will be directly under the nostrils. The cushion will hug the nose and mouth, and leak will be minimal.

Note: At no time should the nose be inside of the cushion opening. If any part of the cushion overlaps the nose, the cushion is being worn incorrectly. A different size cushion may be needed.



2

With the mask assembled, place the cushion under the nose.



3

Position the frame on the top of the head. Pull the headgear over the back of the head.



4

Attach the magnetic headgear clips to the mask cushion.



5

To adjust the mask, peel the headgear tabs away from the fabric. Adjust the length of the top straps. Press the tabs back against the fabric to reattach. Repeat this step with the bottom straps.

Note: Do not overtighten the headgear.



6

Position the mask until it fits comfortably. When finished, the elbow should rest at the top of the head.

Note: Please read the instructions for use for further fitting information.

## Nasal

Using the nasal cushion sizing gauge (included with the mask), hold it horizontally under the nose. Select the cushion size based on where the outer edge of the nostrils and tip of the nose contact the gauge.

Note: Four cushion sizes (small, medium, large, medium-wide) are available. Please refer to your instructions for use for further fitting information.

## Silicone pillows

Using the silicone pillows sizing gauge, place the pillows cushion tips into the nostrils. Choose the cushion size that allows the silicone bases to sit comfortably under the nose without any gaps.

Note: Four cushion sizes (small, medium, large, medium-wide) are available. Please refer to your instructions for use for further fitting information.

1



The cushion opening will be directly under the nostrils. The cushion will hug the nose and leak will be minimal.

Note: At no time should the nose be inside the cushion opening. If any part of the cushion overlaps the nose, the cushion is being worn incorrectly.



2



With the mask assembled, place the cushion under the nose.



3



Position the frame on the top of the head. Pull the headgear over the back of the head.



4



To adjust the mask, peel the headgear tabs away from the fabric. Adjust the straps length. Press the tabs back against the fabric to reattach.

Note: Do not overtighten the headgear.



5



Position the mask until it fits comfortably. When finished, the elbow should rest at the top of the head.

Note: Please read the instructions for use for further fitting information.



# Making it simple to find the right fit

Achieving the right fit with DreamWear is easy

## Medium frame sizing

The medium mask frame will comfortably fit most faces. If the medium frame does not fit a patient's face, a small or large mask frame might better suit your patient's needs.




Medium correct frame




Proper fit

## Large frame sizing

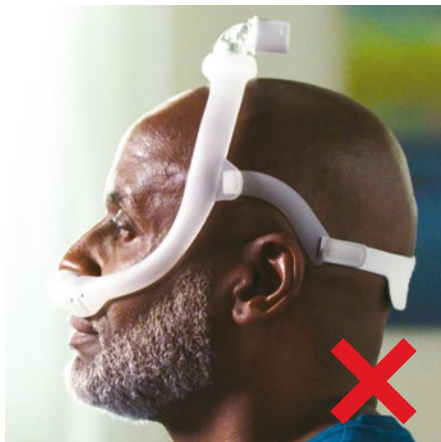

Mask too small

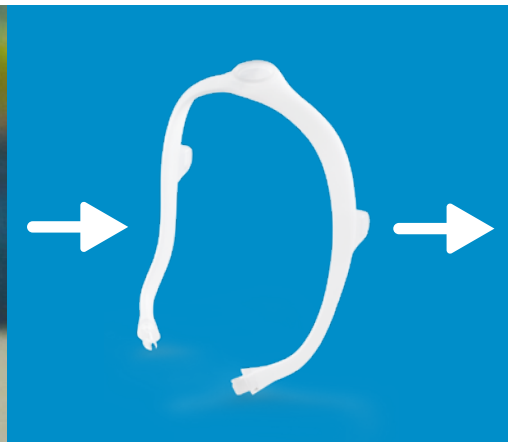


Large correct frame




Proper fit

## Small frame sizing

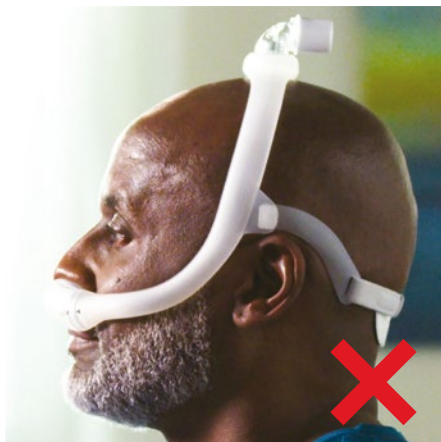

Mask too large

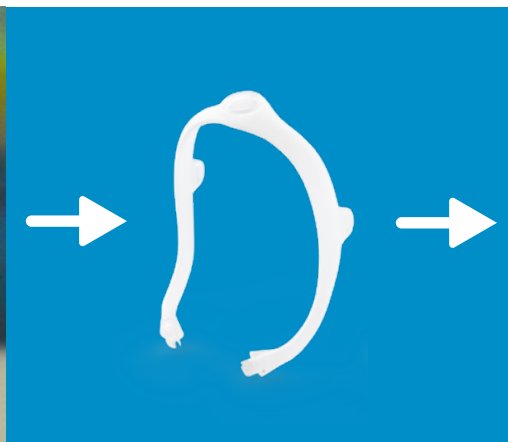

Small correct frame



Proper fit

# Ordering information

|  |                          |                                     |                              |
|--|--------------------------|-------------------------------------|------------------------------|
| <b>DreamWear standard and fitpak configurations</b>                  | <b>Nasal part number</b> | <b>Silicone pillows part number</b> | <b>Full face part number</b> |
| DreamWear with headgear, sm and med frames, small cushion            | N/A                      | N/A                                 | 1133375                      |
| DreamWear with headgear, sm and med frames, medium cushion           | N/A                      | N/A                                 | 1133376                      |
| DreamWear with headgear, sm and med frames, large cushion            | N/A                      | N/A                                 | 1133377                      |
| DreamWear with headgear, sm and med frames, med-wide cushion         | N/A                      | N/A                                 | 1133378                      |
| DreamWear with headgear, med frame, all cushion sizes                | 1116700                  | 1146468                             | 1133400                      |
| DreamWear without headgear, med frame, all cushion sizes             | 1116701                  | N/A                                 | N/A                          |
| DreamWear with headgear arms, sm and med frames, sm and med cushions | 1142376                  | 1146470                             | N/A                          |
| <b>DreamWear mask without headgear, small frame</b>                  | <b>Nasal part number</b> | <b>Silicone pillows part number</b> | <b>Full face part number</b> |
| Small cushion  | 1116710                  | 1146453                             | 1133410                      |
| Medium cushion   | 1116711                  | 1146454                             | 1133411                      |
| Large cushion  | 1116712                  | 1146455                             | 1133412                      |
| Medium-wide cushion*   | 1116713                  | 1146456                             | 1133413                      |
| <b>DreamWear mask without headgear, medium frame</b>                 | <b>Nasal part number</b> | <b>Silicone pillows part number</b> | <b>Full face part number</b> |
| Small cushion  | 1116705                  | 1146457                             | 1133405                      |
| Medium cushion   | 1116706                  | 1146458                             | 1133406                      |
| Large cushion  | 1116707                  | 1146459                             | 1133407                      |
| Medium-wide cushion  | 1116708                  | 1146460                             | 1133408                      |
| <b>DreamWear mask without headgear, large frame</b>                  | <b>Nasal part number</b> | <b>Silicone pillows part number</b> | <b>Full face part number</b> |
| Small cushion  | 1116715                  | 1146461                             | 1133415                      |
| Medium cushion   | 1116716                  | 1146462                             | 1133416                      |
| Large cushion  | 1116717                  | 1146463                             | 1133417                      |
| Medium-wide cushion  | 1116718                  | 1146464                             | 1133418                      |
| <b>DreamWear mask with headgear, small frame</b>                     | <b>Nasal part number</b> | <b>Silicone pillows part number</b> | <b>Full face part number</b> |
| Small cushion  | 1116685                  | 1146410                             | 1133385                      |
| Medium cushion   | 1116686                  | 1146411                             | 1133386                      |
| Large cushion  | 1116687                  | 1146412                             | 1133387                      |
| Medium-wide cushion  | 1116688                  | 1146413                             | 1133388                      |
| <b>DreamWear mask with headgear, medium frame</b>                    | <b>Nasal part number</b> | <b>Silicone pillows part number</b> | <b>Full face part number</b> |
| Small cushion  | 1116680                  | 1146414                             | 1133380                      |
| Medium cushion   | 1116681                  | 1146415                             | 1133381                      |
| Large cushion  | 1116682                  | 1146416                             | 1133382                      |
| Medium-wide cushion  | 1116683                  | 1146417                             | 1133383                      |
| <b>DreamWear mask with headgear, large frame</b>                     | <b>Nasal part number</b> | <b>Silicone pillows part number</b> | <b>Full face part number</b> |
| Small cushion  | 1116690                  | 1146448                             | 1133390                      |
| Medium cushion   | 1116691                  | 1146449                             | 1133391                      |
| Large cushion  | 1116692                  | 1146450                             | 1133392                      |
| Medium-wide cushion  | 1116693                  | 1146451                             | 1133393                      |
| <b>Replacement parts</b>   | <b>Nasal part number</b> | <b>Silicone pillows part number</b> | <b>Full face part number</b> |
| Small cushion  | 1116740                  | 1146472                             | 1133430                      |
| Medium cushion   | 1116741                  | 1146473                             | 1133431                      |
| Large cushion  | 1116742                  | 1146474                             | 1133432                      |
| Medium-wide cushion  | 1116743                  | 1146475                             | 1133433                      |
| Small frame  | 1116745                  | 1116745                             | 1116745                      |
| Medium frame   | 1116746                  | 1116746                             | 1116746                      |
| Large frame  | 1116747                  | 1116747                             | 1116747                      |
| Headgear   | 1116750                  | 1116750                             | 1133450                      |
| DreamWear headgear with arms   | 1116751                  | 1116751                             | NA                           |
| Headgear magnetic clips  | N/A                      | N/A                                 | 1133454                      |
| Fabric wraps   | 1116754                  | 1116754                             | 1116754                      |
| Sizing gauge   | 1116752                  | 1116752                             | 1116752                      |